



AirFit™ N10
Nasal Mask



AirFit™ N10
Nasal Mask

Fitting guide for AirFit N10



- Ensure the grey side of the headgear that shows the ResMed and AirFit N10 logos, is facing outwards.
- Disconnect lower straps. Line up the bottom of the cushion with the bottom of the nose.



- Place the cushion over the nose.
- Hold the mask against the face.
- Pull the headgear up and over the head.



- Bring the lower straps below the ears and attach the EasyClick clips to the outside of the frame.



- Undo the upper fastening tabs.
- Adjust the straps evenly until the mask sits comfortably just under the nose and then reattach the fastening tabs, so that they align with the upper straps.

TIP: The upper straps are the key adjustments for seal and comfort. Ensure they are not over-tightened.



- Undo the fastening tabs on the lower straps. Adjust both of the straps evenly until the mask is stable then reattach tabs.
- Connect the mask tubing to the air tubing attached to the device.

TIP: The lower straps require only minimal tension for the mask to fit comfortably.



- Turn on the device so it blows air.
- Adjust the cushion and headgear until the mask sits comfortably on the nose and is not creased or twisted.
- The mask is now ready to use.

Note: For complete user instructions refer to the User Guide.



Additional options

- The patient can position the lower part of the headgear over or under their hair to suit their hairstyle preference.
- Optional soft sleeves may be assembled for enhanced comfort on the cheeks.

Try one today or find out more at
ResMed.com/AirFitN10
[#AirFitN10](https://twitter.com/AirFitN10)



Effortless performance

ResMed's AirFit™ N10 compact nasal mask is part of ResMed's new AirFit range — the right fit for you. Its excellent performance, combined with optimal usability, brings real innovation to the compact nasal category. The AirFit N10 is simple and intuitive for patients to fit and comfortable to wear, with an under-eye frame providing clear line of sight.

Simplicity

Self-aligning EasyClick clips secure the frame to the headgear easily. Plus it has a simple key-clip cushion and a durable yet highly flexible integrated short tube.

Comfort

The soft cushion, SoftEdge™ headgear and optional soft sleeves all contribute to patient satisfaction. And diffused venting helps create a comfortable environment.

Visual freedom

With a compact design and a sleek frame that's positioned underneath the eyes, the AirFit N10 provides patients with unrestricted vision.

Embrace simplicity

AirFit N10 provides patients with a mask that's easy to use and quick to fit. The uniquely designed EasyClick clips allow for simple fitting and removal without the need to adjust the headgear every time. And AirFit N10 delivers stability without forehead support, providing patients with visual freedom and minimal facial contact. All designed to help improve patient adherence to therapy, AirFit N10 is the right fit for you and your patients.



AirFit N10 features and benefits


SoftEdge headgear design
with premium rolled-edge fabric is comfortable to wear and designed to reduce facial marking.



Key-clip cushion
that's quick and simple to remove and attach.

Circular diffused venting
directs air away for minimal user and partner disturbance.

Lightweight, durable flexi-tube
is integrated into the frame. It's designed to seamlessly stretch and bend 360° to accommodate patient movement during sleep, without compromising mask seal.

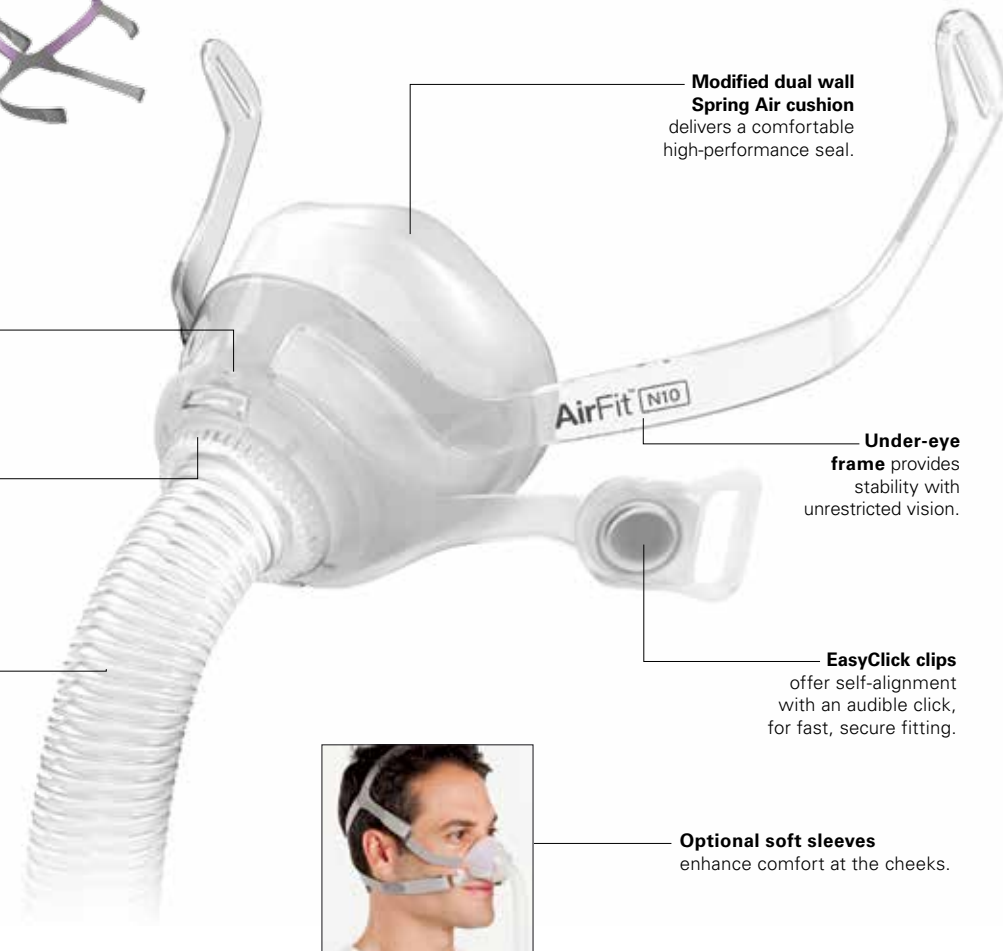

Modified dual wall Spring Air cushion
delivers a comfortable high-performance seal.


Under-eye frame
provides stability with unrestricted vision.

EasyClick clips
offer self-alignment with an audible click, for fast, secure fitting.

Optional soft sleeves
enhance comfort at the cheeks.



AirFit N10 components

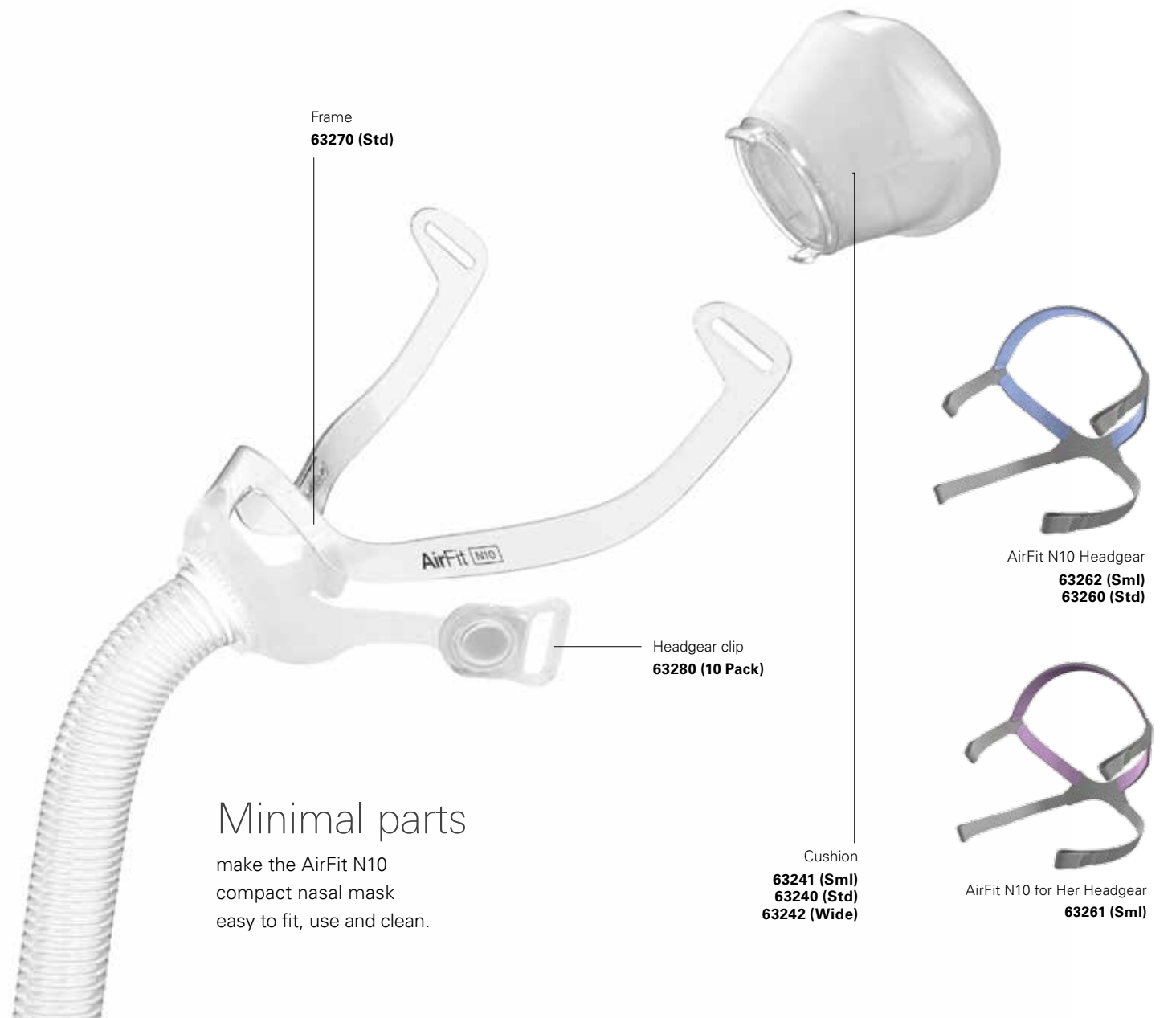

Frame
63270 (Std)

Headgear clip
63280 (10 Pack)

Cushion
63241 (Sml)
63240 (Std)
63242 (Wide)

AirFit N10 Headgear
63262 (Sml)
63260 (Std)

AirFit N10 for Her Headgear
63261 (Sml)

Minimal parts

make the AirFit N10 compact nasal mask easy to fit, use and clean.

Product Codes

	ANZ
AirFit N10 for Her mask system Small	63219
AirFit N10 mask system Standard	63218
AirFit N10 mask system Wide	63220

Note: For more details, refer to the User Guide.

AirFit™ N10
Nasal Mask