OPTICAL FRAMES

(ACETATE, TITANIUM, TITANIUM-ACETATE)

INSTRUCTIONS FOR USE

Salt Optics, Inc. "SALT"

SALT. Optical Frames INSTRUCTIONS FOR USE

Contact information:

Web: https://saltoptics.com/collections/optical

Customer Service: <u>SERVICE@SALTOPTICS.COM</u>

US Phone: +1 949 574 3800

Europe Phone: +33 4 93 84 24 29



Salt Optics, Inc.

1607 Monrovia Ave. Costa Mesa, CA 92627 USA



MedEnvoy Global B.V.

Prinses Margrietplantsoen 33 Suite 123 2595 AM The Hague The Netherlands



MedEnvoy Switzerland

Gotthardstrasse 28 6302 Zug Switzerland

UK Responsible Person MedEnvoy UK Limited

85, Great Portland Street First Floor London, W1W 7LT United Kingdom



Table of Contents:

PRODUCT DESCRIPTION	3
INTENDED USE	3
INDICATIONS FOR USE	3
INTENDED CLINICAL BENEFIT	3
CONTRAINDICATIONS AND WARNINGS	3
BASIC OPERATION	3
PREVENTION AND CARE	3
TRANSPORT AND STORAGE	3
EXPLANATION OF SYMBOLS USED IN LABELLING	4
WARRANTY	5
SERVICE AND REPAIR	5
CERTIFICATE OF CONFORMITY	5

PRODUCT DESCRIPTION

SALT Optical Frames are optical eyeglass frames constructed out of acetate, titanium, or a combination thereof. The frames are provided with non-prescription lenses that are designed to be removed and replaced with prescription lenses by a licensed optician or doctor before use by the patient.

INTENDED USE

SALT Optical Frames are a device intended to hold prescription spectacle lenses worn by the patient to correct refractive errors.

INDICATIONS FOR USE

Completed eyeglasses are used to correct or improve the vision of people with nearsightedness (myopia), farsightedness (hyperopia), presbyopia, and/or astigmatism.

INTENDED CLINICAL BENEFIT

Improve the function of the eyes (vision).

CONTRAINDICATIONS and WARNINGS

WARNING: Do not use as safety glasses.

MARNING: Avoid high impact.

/ WARNING: A healthcare professional should be consulted to adjust the frames to the face.

BASIC OPERATION

- 1. Slide the arms of the eyeglass frames over your ears and lower the frame gently onto your nose.
- 2. Handle your glasses with two hands to reduce strain on the hinges.

PREVENTION AND CARE

- 1. Use an eyeglass case.
- 2. Clean your glasses on a regular basis, always with a microfiber cloth or high-quality cotton.
- 3. Remove your eyeglasses before going to sleep.
- 4. Remove your eyeglasses when you are practicing sports.
- 5. Have your eyeglass frames adjusted at least once a year.


TRANSPORT AND STORAGE

When not in use, keep in the cloth or hard case in a dry place at a temperature between 10 to 40 degrees centigrade.

For safe transportation, the device should be repacked in the provided hard-shell carry case.

Should it be necessary to return the device to the manufacturer, please contact the manufacturer first.

EXPLANATION OF SYMBOLS USED IN LABELLING

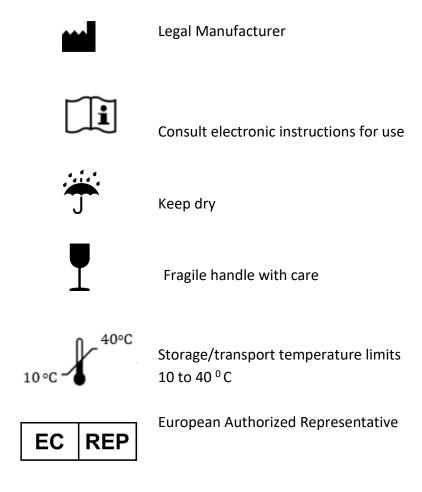

WARRANTY

North American Customers:

- SALT extends a one-year defective warranty from the date of purchase on all eyewear. Eyeglass
 frames may be repaired or replaced upon inspection for any factory defect, provided any damage
 cannot be attributed to normal wear and tear. Accidental breakage or scratched lenses are not
 covered by warranty, but such repairs may be serviced for a standard fee.
- Warranty Requirement:
 - o Original sales receipt or proof of purchase, including purchase date and retailer information.
 - o An explanation of the frame defect or how the frame was damaged.
 - Your full contact details including the following: name, address, telephone number, as well as your email address.
 - Proper return address
 - Shipping paid in full. We do not accept returns on cod.
 - All returns must be accompanied by a Return Authorization (RA) number. Please contact SALT to obtain the RA number. Packages without a return authorization number will be returned back to sender.

European Customers:

- As required by law, SALT. extends a two-year defective warranty from the date of purchase on all
 eyewear. Eyeglass frames may be replaced upon inspection for any factory defect, provided any
 damage cannot be attributed to normal wear and tear. Accidental breakage or scratched lenses are
 not covered by warranty.
- Please contact your local SALT Retailer for any warranty claims.
- Warranty Requirement:
 - o Original sales receipt or proof of purchase, including purchase date and retailer information.
 - An explanation of the frame defect or how the frame was damaged.

For All Other Countries:

• Please contact SALT. directly.

SERVICE AND REPAIR

For servicing information or repair information, please contact SALT.

EU: +33 4 93 84 24 29 US: +1 949 574 3800

EU/US: RETURNS@SALTOPTICS.COM

CERTIFICATE OF CONFORMITY

The EU Declaration of Conformity is available at https://saltoptics.com

SALT Optics, Inc.	Document number		Issue	1
	Title	SALT Optical Frames – Instructions for Use		r Use

Document Title	SALT Optical Frames – Instructions for Use		
Issue No	1	Issue Date	17 March 2023

APPROVALS			
Role	Name	Position	Date
Author	Marielle Lejcher	Regulatory Consultant	17 March 2023
Approver	Jaysen Johnson	Financial Controller	17 March 2023

AMENDMENT RECORD			
Date of Amendment	Document Revision	Pages Affected	Reason for Amendment
12/03/2018	01	All	Initial MDR Release