# ***Instant Ear Thermometer***

Model 09-340

**Please read these instructions completely before  
using your Instant Ear Thermometer**

**INSTRUCTION MANUAL • ENGLISH & ESPAÑOL**

---

## TABLE OF CONTENTS

---

Introduction .....	3
Quick Tips.....	4
Product Features .....	5
Symbols .....	5
Care and Safety Information .....	8
Cleaning and Storage .....	9
Taking a Reading.....	6
Changing Modes.....	7
Recalling Readings in Memory.....	7
Replacing the Battery .....	7
Troubleshooting.....	10
Product Specifications.....	11
Warranty Information .....	12
Español.....	13-24

---

**Toll-Free Customer Care Help Line: 1-866-326-1313**  
**Monday – Friday 8:30 a.m. – 4:30 p.m. CST**

Distributed by  
Veridian Healthcare  
1175 Lakeside  
Gurnee, IL 60031  
[www.veridianhealthcare.com](http://www.veridianhealthcare.com)  
Made in China  
#93-1041 12/12  
©2012 Veridian Healthcare, LLC

---

# INTRODUCTION

---

Thank you for purchasing a Veridian Healthcare Instant Ear Thermometer. Please read this instruction manual completely before using your unit, to ensure safe use, accurate results and long life of the thermometer.

Ear thermometers (also known as tympanic or aural thermometers), are a great alternative to oral, rectal and underarm thermometers. The ear is an excellent site for measuring temperature, as the part of the brain that regulates body temperature shares a blood supply with the eardrum. This thermometer provides results several times faster than the fastest oral thermometer, making it ideal for infants and fidgety children. It can even be used while the patient is sleeping. Unlike readings from oral thermometers, factors such as smoking, drinking, or speaking do not affect the measurements taken by ear thermometers.

The Instant Ear Thermometer takes 512 readings in one second, then converts those data to a single peak temperature, which appears on the display. With proper care, your new thermometer will provide your entire family with many years of safe, accurate and comfortable temperature readings.


---

## QUICK TIPS

---

- Ear temperature readings are equivalent to oral temperature readings.
- Readings taken while asleep should not be compared directly to readings taken while awake, as body temperature while asleep is typically lower.
- Always take temperature readings from the same ear, as ears may differ in temperature.
- Wait at least 20 minutes after bathing before taking a temperature reading to ensure the most accurate results.
- The best position for infants when taking a reading is lying flat, with the head to the side so the ear being used faces the ceiling.
- For adults and children, a standing position is best. The person taking the reading should stand behind and slightly to the side.
- For certain people, it is best to take three consecutive readings (waiting at least 10 seconds between each), using the highest temperature as the reading. These populations include:
  - Infants less than 100 days old
  - Children under three years with compromised immune systems, or in cases where the monitoring of temperature is important
  - New users of the Instant Ear Thermometer, who are unfamiliar with the technique
- Normal temperature readings can vary from 97.0° F to 99.0° F (36.1°C to 37.2° C). Level of activity, time of day, and other factors may affect body temperature.

# PRODUCT FEATURES AND SYMBOLS

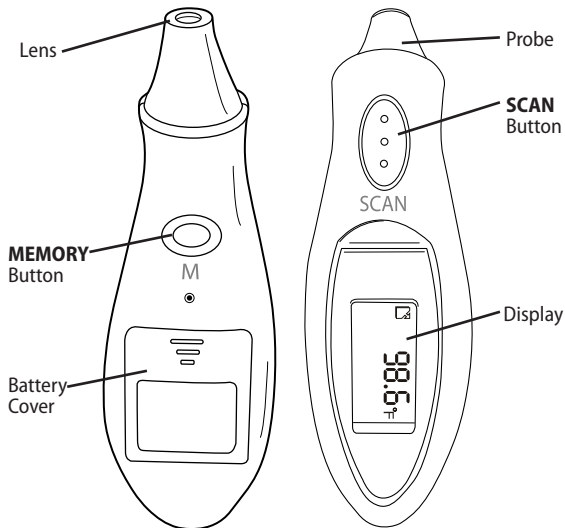Scan in progress		Fahrenheit/Celsius °F / °C	
Temperature below 100.4° F		Low battery	
Temperature 100.4° F or higher		Recalled memory reading	

---

## TAKING A READING

---


1. Clean the lens at the tip of the thermometer, using alcohol and a soft cotton swab. Cleaning the lens before each use ensures the most accurate reading and prevents cross-contamination, see page 9 for complete cleaning and care instructions.
2. Press and release the red "SCAN" button. The thermometer will perform a brief self-test, during which all indicators on the display will be shown, *Fig. 1*, followed by the last reading taken.
3. The thermometer will beep, indicating that the unit is ready for measurement. Gently insert the probe into the ear canal until a slight resistance is felt, *Fig. 2*.
4. Press and release the SCAN button again, while keeping the probe in place. A beep will sound when the measurement is complete.
5. The temperature reading will be shown on the display.
6. If the temperature measurement is below 100.4° F, the beep indicating the reading is complete will be a single, short beep. A happy face will be displayed next to the reading, *Fig. 3*.
7. If the reading is 100.4° F or above, the unit will sound a long beep, followed by two short beeps. A sad face will be displayed, *Fig. 4*.
8. After about one minute after use, the thermometer will automatically beep and shut off.
9. Clean the thermometer lens again after use. Wait at least 10 seconds between consecutive readings, to ensure accuracy.

*Fig. 1*

*Fig. 2*

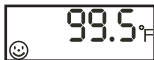*Fig. 3*

*Fig. 4*

**NOTE:** The backlit display will remain lit for approximately 5 seconds each time the SCAN button is pressed, after each reading is completed, and each time the Memory (M) button is pressed to recall a previous reading.

---


## CHANGING MODES

---

Your thermometer can display results in either degrees Fahrenheit (°F) or degrees Celsius (°C). To switch between Fahrenheit and Celsius, press and hold the SCAN button. While holding the SCAN button, press the Memory (M) button to toggle between modes. *Figs 1, 2.*

*Fig. 1*

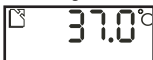
*Fig. 2*

---

## RECALLING READINGS IN MEMORY

---

Your Instant Ear Thermometer automatically stores the last 10 readings taken. To view the readings stored in memory, press the SCAN button to turn the thermometer on and wait for the beep. You may then press the Memory button to recall the most recent measurement. Press the Memory button multiple times to view previous readings (the unit will beep each time you switch readings). The index number to the left of the temperature reading indicates the sequential number of the stored reading you are currently viewing, *Fig. 1.*

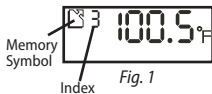*Fig. 1*

---

## REPLACING THE BATTERY

---

Your thermometer came with one lithium battery (type CR2032) installed. When the low battery symbol appears,  the battery needs to be replaced.

**Replacing the battery, *Fig. 1:***

1. Slide the battery cover off
2. Carefully remove the old battery. Do not use a metal-pointed tool.
3. Insert the new battery with the positive (+) side facing up.
4. Replace the battery cover
5. Dispose of the old battery properly at an appropriate facility. Keep the battery away from children, sunlight and heat.

*Fig. 1*

---

## CARE AND SAFETY INFORMATION

---

Improper care or use of your thermometer may result in damage to the unit or inaccurate readings, which can lead to a misdiagnosed fever. Following these instructions will ensure the thermometer's accuracy and long life.

- Never allow the thermometer to be used without adult supervision.
- Do not use excessive force to insert the probe into the ear; proper use should not cause pain or discomfort.
- The ear should be clean before using this thermometer; earwax in the ear canal may cause an incorrect reading. For proper ear cleaning techniques, consult a healthcare professional.
- Do not use the thermometer if the ear is infected or the ear canal is wet.
- The patient should remain still and avoid talking while the thermometer is in use.
- Clean the lens before and after each use.
- Do not touch the lens with fingers.
- Store the thermometer in the storage case when not in use.
- Do not store the thermometer in direct sunlight, dusty or humid environments, or extreme temperatures. Exact figures for appropriate use and storage can be found in the Product Specifications section of this manual.
- If the patient's temperature is below the ambient air temperature, an inaccurate reading may result.
- Dropping the thermometer or causing other heavy impact may damage it.
- Do not disassemble the thermometer or attempt to repair it (other than replacing the battery).
- If a reading cannot be taken, and the troubleshooting steps do not fix the problem, the thermometer must be replaced.

---

## CLEANING AND STORAGE

---


The lens should be cleaned before and after each use, to eliminate earwax and fingerprints and ensure the accuracy of your readings. Proper storage will prevent degradation and damage to the unit.

### CLEANING:


- An alcohol swab, or a soft cotton swab moistened with 70% isopropyl alcohol, should be used to gently cleanse the probe lens and thermometer casing, *Fig. 1-3*.
- Never use a paper towel to clean your thermometer, as it may scratch the lens.
- Do not immerse the thermometer or allow liquid to enter the casing.
- Do not use cleaning agents other than alcohol to clean the thermometer; abrasives, thinners or gasoline will damage the unit.

### STORAGE:


- Always store the thermometer in the provided storage case.
- Protect the thermometer from mechanical shock or heavy impact.
- Store the unit in an area away from sunlight, dust and humidity.
- Keep the thermometer in an environment between 68° F and 122° F (20° C - 50° C).
- Remove the battery during extended storage.

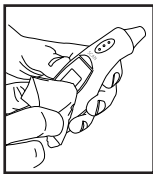*Fig. 1*

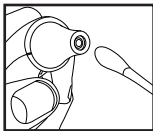*Fig. 2*

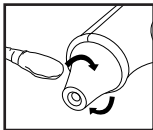*Fig. 3*

---

## TROUBLE SHOOTING

---

The following chart indicates error messages that may appear on your display, why they appear, and how you may solve the problem.

ERROR MESSAGE	PROBLEM OR CAUSE	SOLUTION
	Low battery warning	Replace battery when possible (thermometer will work)
	Low battery	Replace battery immediately
	Unit turned off or battery problem	Press SCAN button to activate unit. Check battery type or polarity (position in the unit). Replace battery.
	Temperature measurement is below typical temperature range	Gently reposition the probe firmly into the ear canal and try again.
	Temperature measurement is above typical temperature range	Gently reposition the probe firmly into the ear canal and try again.
	Measurement is below the normal temperature range	Reposition the probe firmly into the ear canal and try again.

---

# PRODUCT SPECIFICATIONS

---

<b>Name</b>	Instant Ear Thermometer
<b>Model number</b>	09-340
<b>Display system</b>	Liquid Crystal Display with 0.1° F (0.1° C) resolution
<b>Accuracy</b>	+/- 0.4° F, 96.8° F – 102.2° F (+/- 0.2° C, 36.0° C – 39° C) +/- 0.5° F, <96.8° F or >102.2° F (+/- 0.3° C, <36.0° C or >39° C)
<b>Measuring Range</b>	93.2° F – 109.4° F (34.0° C – 43.0° C)
<b>Memory</b>	Automatically stores up to 10 readings.
<b>Tone</b>	One short beep when the unit is turned on Once the measurement is complete: <ul style="list-style-type: none"><li>• One beep sounds for temperatures under 100.4° F (38.0° C)</li><li>• One long beep followed by two short beeps sound for temperatures above 100.4° F (38.0° C).</li><li>• One short beep sounds for temperatures &lt;93.2° F (34° C) (Lo) or &gt;109.4° F (43° C) (Hi).</li></ul> Beeps once for each memory recall Beeps once when unit automatically turns off Short rapid continuous beeping indicates battery is low
<b>Operating temperature</b>	60.8° F to 104.0° F (16.0° C to 40.0° C) at relative air humidity (not condensing) or up to 95%.
<b>Storage temperature</b>	-4.0° F to 122.0° F (-20.0° C to 50.0° C)
<b>Automatic power-off</b>	Automatic shut-off after approximately 1 minute of non-use.
<b>Battery</b>	1 each, CR2032 battery
<b>Dimensions</b>	4-1/2" (L) x 1-3/8" (W) x 1" (H) (11.5 cm x 3.5 cm x 2.5 cm)
<b>Weight</b>	With battery, 1.6 ounces (45 grams)
<b>Accessories</b>	Storage case, Instruction Manual, Quick Start Guide, Frequently Asked Questions, one CR 2032 battery
<b>Standards</b>	Complies with US ASTM E-1965-98.

*Specifications are subject to change without notice.*

---

---

# WARRANTY INFORMATION

---

## LIMITED LIFETIME WARRANTY

The warrantor guarantees that its Instant Ear Thermometer will be free from manufacturing defects under normal use for as long as the original purchaser owns the product. This warranty covers only normal use, and does not apply to use in any clinical or commercial applications. This warranty does not cover batteries or other power sources that may be provided with or used with the Ear Thermometer. This warranty is voided if the Ear Thermometer product is misused or abused in any manner.

If the Instant Ear Thermometer fails to operate during the time the original purchaser owns it, return it postage prepaid to: **Repair Department, 1175 Lakeside Drive, Gurnee, IL 60031**. The warrantor will repair or replace the defective unit at the option of the warrantor. Repair or replacement of the defective unit, at warrantor's option, is the sole remedy under this warranty.

ANY IMPLIED WARRANTIES WHICH THE PURCHASER MAY HAVE ARE LIMITED IN DURATION TO THE TIME THAT THE ORIGINAL CONSUMER PURCHASER OWNS THE PRODUCT. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

This warranty constitutes the warrantors only responsibility and obligation to repair and/or replace materials or components, or refund the purchase price. The warrantor will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss, including, but not limited to, damage to or loss of other property or equipment and personal injuries, whether to purchaser or others. The warrantor shall in no event be liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchase price of the unit. Some states do not allow the exclusive or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

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