


IMPORTANT: Preparing for First Use


Must boil these parts (unassembled) for 10 minutes for secure fit between parts. (Before first use only)

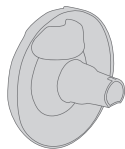

Breast Funnel



Valve Base

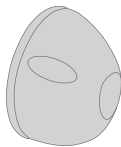

Cup


Ensure all parts are dry prior to use. Do not steam sterilize or use abrasives on any Freemie components. Steam may damage parts and cause these parts to not fit securely.

Clean parts with hot soapy water, rinse thoroughly after each use. Do not store assembled.


See reverse side →

IMPORTANT: Proper Milk Barrier Assembly


Incorrect Assembly of the milk backflow barrier will cause low or no suction.

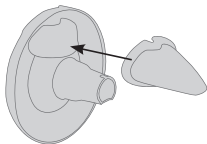

Step 1: Place Barrier onto funnel mounting ridge and against cone surface.

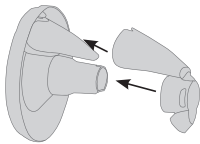


Step 2: Align Valve Base with Barrier and Funnel and push together.

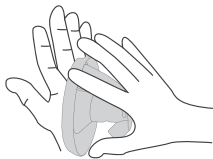


Step 3: Press Breast Funnel and Valve Base firmly together.

IMPORTANT: Proper Cup Alignment

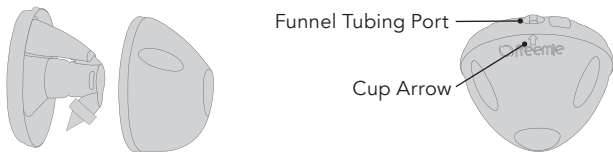


Align cup arrow with funnel tubing port, seat cup rim into the outer ring of the funnel, and snap together to secure the assembly.

IMPORTANT: Inserting Tubing into Collection Cone

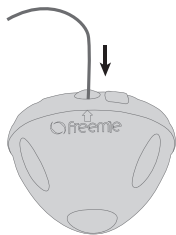

Insert tubing into funnel tubing port until a secure connection is achieved.
(1/4 - 1/2 in, 9-13 mm)

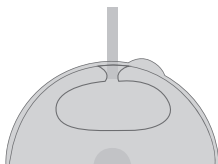

Detached or loose tubing connection will cause low or no suction.