



Southwest Medical Refurbishing Process for Hospital Beds

- ***Run Serial Number With OEM For Necessary Upgrades, Recalls, And Revisions***
- ***Disassemble And Disinfect All Components***
- ***Polish, Replate, Repaint, Or Replace All Exterior Components***
- ***Inspect All Working Components For Wear And Replace Defective Parts As Necessary***
- ***Replace Wheel And Caster Assembly***
- ***Replace Safety Switches***
- ***Replace Lockout Switches***
- ***Replace Foot Pedals If Necessary***
- ***Inspect, Clean, And Lubricate HEAD, KNEE, And HIGH/LOW Drive Screws For Reuse***
- ***Inspect Nurse Control, HIGH/LOW, And Degree Indicator Panels***
- ***Recalibrate Brake And Steer System***
- ***Reassemble Refurbished Components***
- ***Inspect And Lubricate All Pivot Points***
- ***Inspect And Lubricate Instant CPR Cables And Latch Release (If Applicable)***
- ***Inspect And Lubricate Siderail Latches***
- ***Replace Siderail Plastics***
- ***Replace Laminate In Headboard/Footboard With Custom Finish Of Customer Choice***
- ***Stage Test Weight Tolerances And Table Movement Through All Available Articulations***
- ***Apply New Decals And Safety Information***

Call or Email Southwest Medical for Additional Information

(800) 259-5940

www.swmedical.com