

## Measurement Guide

FOR FIDOGEAR CART HARNESS

### WHAT YOU WILL NEED

1. Fabric Tape Measure

2. Writing Utensil

3. Treats (for The Dawg) Get some for you as well, just in case you get famished in the process



#### TIPS WHILE MEASURING

We always suggest having a partner assist you while measuring your dog. The partner can distract your dog with treats/toys and keep their attention while you measure. Also, your dog must be standing straight and looking forward (also known as standing "square"). Having the dawg "Square" keeps your measurements consistent and accurate.

### When using the Treats:


- 1. It works best to hold a treat between your fingers, do not let "The Dawg" get the treat right away, have them work at your fingers a bit, giving them treats sporadically.
- 2. The treats for you, can be consumed whenever and as often as you like. Whatever will keep you happy during this process.

#### When using the Tape Measure:



- 1. Be sure to pull the tape measure snug against "The Dawg's" body, as this will create a proper fit. This is especially important for Dawg's with heavier coats.
- 2. When going from "Point to Point" try to keep your fingers on each point. For instance, when measuring "Point A to B, then Point A to C" try to keep your finger on "Point A"

when doing both of those measurements.

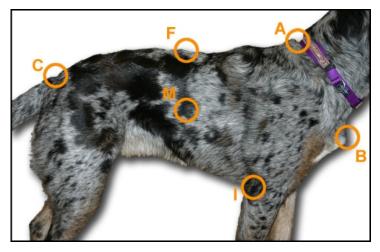

#### NOTE:

The dog model in this photo is "Sadie" - (AKA -Mascot Dawg). This photo is for reference, depending on the breed, size, sex, etc. These points can vary slightly on location.

Sadie is also, not standing "Square" in this image, her right front leg would preferably be forward a bit more than shown.

### **POINT DESCRIPTIONS**


- (A) Back of neck at withers (I) Deepest point of
- (B) Point of Breastbone
- Chest behind Elbow
- (C) Base of Tail
- (M) Center of Dog's side

(F) Middle of Back




# Measurement Guide


FOR FIDOGEAR CART HARNESS - CONT. PG 2

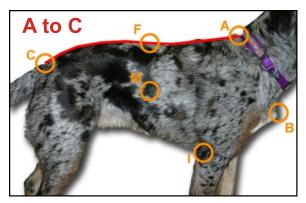


Measure (A) - Base of neck at withers to (C) - Base of Tail. To see (C) better, try gently lifting the dogs tail to see where the tail meets the dogs body - this is Point (C)

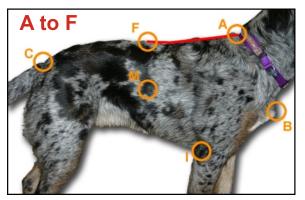


Measure (A) - Base of neck at withers to (F) - Middle of Back.

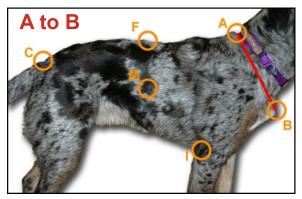


Measure (A) - Base of neck at withers to (B) - Point of breastbone, when you feel for the breastbone, roll your finger slightly up to the top of the bone. This is Point (B)

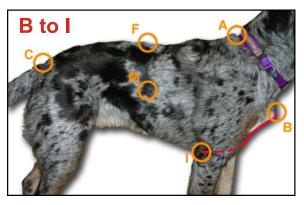

Measure (B) - Point of breastbone to (I) - Deepest Point of Chest behind Elbow, when you feel for the breastbone, roll your finger slightly up to the top of the bone. Point (I) can just barely be seen from side of dog. Dotted line indicates going between front legs.

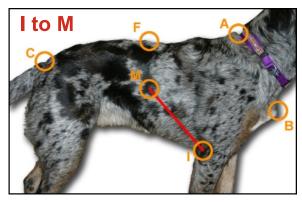

Measure (I) - Deepest Point of Chest behind Elbow to (M) - Center of Dog's Side Hold your finger at (M) and move to next measurement

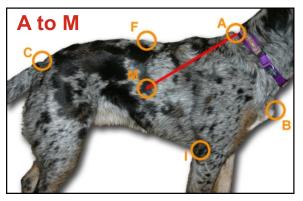


By holding your finger at (M) you help ensure the point stays the same. Measure (A) - Base of neck at withers to (M) - Center of Dog's Side




# Measurement Guide


FOR FIDOGEAR CART HARNESS - CONT. PG 3

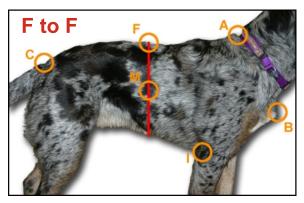

Measure (F) - Middle of Back to (F) Belly Circumference

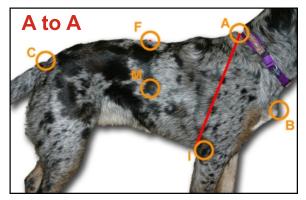

Measure (A) - Base of Neck at Withers to (I) - Deepest Point of Chest behind Elbow, and back up to (A) = (A to A) Chest Circumference

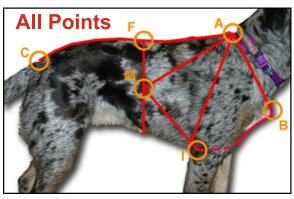

Here is a briefing as to all the measurements you just did.